

NAMADA-Dx™

CASSETTE AT-HOME PREGNANCY TEST

IVD

WARNING:

- Do not use after expiration date (date on plastic foil cover).
- Keep test stored at 4-30°C
- Do not store the test in a freezer
- Keep away from children
- Do not remove test device from pouch until ready to perform test
- For external use only.

Before use read instructions carefully

KIT CONTENTS:

- Instructions for use
- Test device
- Dropper
- Desiccant
(to be discarded after opening the pouch)



TEST PROCEDURE

Collect urine in a clean and dry container. Remove the test device from pouch and put it on a flat surface. Hold the dropper vertically and transfer 3 full drops of urine to the specimen well (S) of the test device, and then begin timing (3 minutes). Avoid trapping air bubbles in the specimen well (S). See the illustration below.


Wait for the red line (s) to appear. It is important that the background is clear before the result is read.

NAMADA-Dx™

CASSETTE AT-HOME PREGNANCY TEST

READING THE RESULTS:

The results should be read after three minutes. If two red lines, appear in the result window that means you are pregnant (the intensity of the red is insignificant). If only one red line appears in the control region, you are not pregnant. If no line appears at all, that means that the test did not work properly. Repeat the test using a new test pack. Follow the directions very carefully.

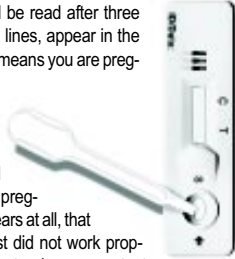

Positive



Negative



Invalid result

Remember: A proper diet as well as abstinence from smoking and alcohol can increase your chances for a healthy pregnancy.

NAMADA-Dx™

CASSETTE AT-HOME PREGNANCY TEST

QUESTIONS AND ANSWERS:

Q. When can pregnancy detected?

A. Pregnancy can be detected from a very early stage, about 10 days after fertilization of the ovum, 2-3 days before the delayed period (when the menstrual cycle is regular). When the test is carried out exactly according to the instructions, the accuracy is very high, similar to a urine pregnancy test carried out in a medical laboratory (99% accuracy).

Q. Should first morning urine be used?

A. Before a missed period we recommend that first morning urine be used. After a missed period any urine may be used.

Q. If the results are negative but the menstrual period has not occurred what should be done?

A. In this case we advise to carry out the test again with a new **mamatest** after 3 days. If this test also gives a negative result, a physician should be consulted to find out the reasons for the delay. In case of pain or unexpected bleeding a physician should be consulted.

Q. Can certain drugs alter the test results?

A. Prescribed drugs containing hCG in the form of an injection could influence the result. Birth control pills, painkillers and antibiotics have no effect on the test results.

Q. What should I do if the result is positive?

A. You should see your doctor as soon as possible. Remember that the first three months of pregnancy are the most important ones.

NAMADA-Dx™

CASSETTE AT-HOME PREGNANCY TEST

Q. When should I take the test if my periods are irregular?

A. In this case estimate when the period should start by taking into account the longest cycle you've had in the last few months. Take the test on that day. If the results show that you are not pregnant, and your period still does not begin, repeat the test after three days. If you do not begin to menstruate contact your doctor.

Q. Is the amount of time I wait to read my test important?

A. Yes. You should read the results after three minutes. Depending on the concentration of hCG hormone, the positive result could show between one and three minutes. After ten minutes, the results could be inaccurate.

After reading the results discard the test



AZOG, Inc.

1011 US HWY 22

Phillipsburg, NJ 08865, USA

QUESTIONS?

Call toll-free 1-866-865-4881

M-F, 8:30am-5pm EST

e-mail: info@azoglabs.com,

www.azoglabs.com