

NAMADA-Dx™

MIDSTREAM AT-HOME PREGNANCY TEST

IVD

WARNING:

- Do not use after expiration date (*date on plastic foil cover*).
- Keep test stored at 4-30°C
- Do not store the test in a freezer
- Keep away from children
- Do not open the plastic foil cover if you do not intend to use test within 4 hours
- For external use only.

Before use read instructions carefully

KIT CONTENTS:

- Instructions for use
- Pouched test
- Desiccant
(to be discarded after opening the pouch)



TEST PROCEDURE:

Remove the Cap and expose the Absorbent tip. Hold the midstream test with the exposed Absorbent. Tip pointing downward directly into your urine stream for at least 10 seconds until it is thoroughly wet.

Note: **Do not urinate on the Test and Control windows.** If you prefer, you can urinate into a clean and dry container, then dip only the Absorbent Tip of the midstream test into the urine for at least 10 seconds.

After removing the midstream test from your urine,

NAMADA-Dx™


MIDSTREAM AT-HOME PREGNANCY TEST

immediately replace the Cap over the Absorbent Tip, lay the midstream test on a flat surface with the Test and Control windows facing upwards and then begin timing.



READING THE RESULTS:

The results should be read after three minutes. If two red lines, appear in the result window that means you are pregnant (the intensity of the red is insignificant). If only one red line appears in the control region, you are not pregnant. If no line appears at all, that means that the test did not work properly. Repeat the test using a new test pack. Follow the directions very carefully.

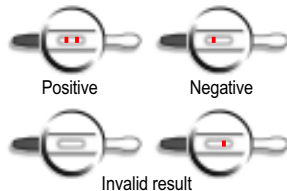

Remember: A proper diet as well as abstinence from smoking and alcohol can increase your chances for a healthy pregnancy.

NAMADA-Dx™

MIDSTREAM AT-HOME PREGNANCY TEST

QUESTIONS AND ANSWERS:

Q. When can pregnancy detected?

A. Pregnancy can be detected from a very early stage, about 10 days after fertilization of the ovum, 2-3 days before the delayed period (when the menstrual cycle is regular). When the test is carried out exactly according to the instructions, the accuracy is very high, similar to a urine pregnancy test carried out in a medical laboratory (99% accuracy).

Q. Should first morning urine be used?

A. Before a missed period we recommend that first morning urine be used. After a missed period any urine may be used.

Q. If the results are negative but the menstrual period has not occurred what should be done?

A. In this case we advise to carry out the test again with a new **mamatest** after 3 days. If this test also gives a negative result, a physician should be consulted to find out the reasons for the delay. In case of pain or unexpected bleeding a physician should be consulted.

Q. Can certain drugs alter the test results?

A. Prescribed drugs containing hCG in the form of an injection could influence the result. Birth control pills, painkillers and antibiotics have no effect on the test results.

Q. What should I do if the result is positive?

A. You should see your doctor as soon as possible. Remember that the first three months of pregnancy are the most important ones.

NAMADA-Dx™

MIDSTREAM AT-HOME PREGNANCY TEST

Q. When should I take the test if my periods are irregular?

A. In this case estimate when the period should start by taking into account the longest cycle you've had in the last few months. Take the test on that day. If the results show that you are not pregnant, and your period still does not begin, repeat the test after three days. If you do not begin to menstruate contact your doctor.

Q. Is the amount of time I wait to read my test important?

A. Yes. You should read the results after three minutes. Depending on the concentration of hCG hormone, the positive result could show between one and three minutes. After ten minutes, the results could be inaccurate.

After reading the results discard the test



AZOG, Inc.

1011 US HWY 22

Phillipsburg, NJ 08865, USA

QUESTIONS?

Call toll-free 1-866-865-4881

M-F, 8:30am-5pm EST

e-mail: info@azoglabs.com,

www.azoglabs.com